Nylon - Larga Duración / Nylon - Long Life  
 Neopreno / Neoprene  
 Toalla / Towel

TL165

TL165

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LONG THUMB WRIST SUPPORT WITH PLATE

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LONG THUMB WRIST SUPPORT WITH PLATE

TL165 S	14 - 17 cm
TL165 M	17 - 20 cm
TL165 L	20 - 23 cm

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LONG THUMB WRIST SUPPORT WITH PLATE

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### **GB** Long Thumb Wrist Support with Plate TL165

**ACTIONS:** Provides compression, heat and improves the user's proprioceptive ability.

**INDICATIONS:** Rhizarthrosis, arthritis, tendinitis, bursitis, sprains, mild articular traumas.

**CAUTIONS:** • The doctor is the person qualified to prescribe and decide the duration of the treatment. The orthopaedic technician is the person prepared to fit the product. For maximum effectiveness, you should respect their indications and their instructions of use. • For a correct function, you should use the product correctly adjusted and the correct size. • If the product includes inserts, they should immobilize the painful area. • In the pressure points, the skin should not be injured or be hypersensitive. If you notice any side effects, you should immediately contact your doctor. You should inform your doctor if you suffer of some skin affections. • You should avoid the use of creams in contact with the orthosis during the treatment. • The correct use of the brace depends on the state of all of its components (verify lack of breakages, damages, etc.), so the

brace should be frequently checked by your doctor or healthcare professional, who would indicate the suitability of the product or the convenience to replace it, if the product is damaged. • This orthosis is designed for use on one patient. Do not reuse on other patient or other use than the one prescribed. • Do not allow children to play with this device. • It is advisable to take care of the cleaning of the velcro quick release closures in order to keep its closure characteristics, removing threads, fluffs, etc which may be adhered on them. Clean periodically the product and maintain adequate personal hygiene. • Do not wear the brace when bathing or having or shower, etc.

**CARE INSTRUCTIONS:** • Close all the velcros closures. Handwashable. Do not bleach. Do not tumble dry. Do not iron. Do not dry clean. • Remove the removable components (plate) before washing the brace.

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Prim, S.A.